

apollo[®]

Medial Suture Anchor

Surgical
Technique
Guide




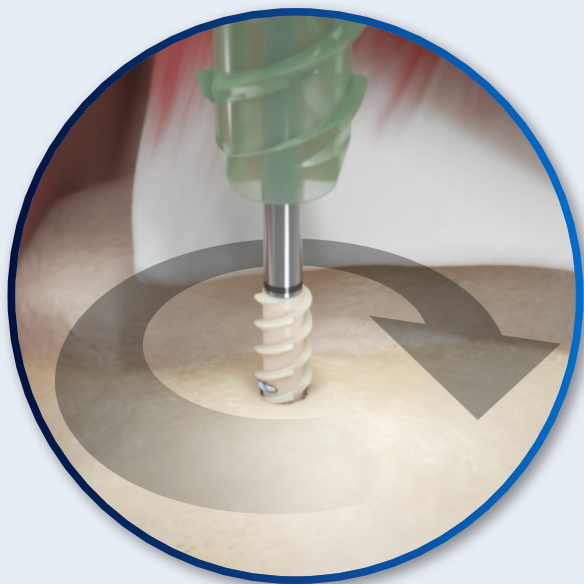
 maruho
medical

Instructions

Punch

Select the corresponding punch/tap for the desired anchor size. To create a pilot hole, gently mallet the punch until the first laser line (15mm) is completely into bone.

 For hard bone, a tap is recommended to create a pilot hole.



Insert

Introduce the apollo Medial Anchor into the prepared hole, engage the threads and **ROTATE THE DRIVER CLOCKWISE** until the laser line is completely into bone.



Release

Release the apollo Medial anchor by pulling straight back on the driver. Suture is now available to finish the repair.



200 Cobb Pkwy N
Building 200, Suite 210
Marietta, GA 30062
888-404-3980
maruho-medical.com



www.maruho-medical.com